

Instructions for Using this Kit

1. Complete all the relevant sections on the Alcohol and/or Drug Determination form. Failure to complete the following sections may delay or prevent testing.
 - a. Subject Information
 - b. Officer's Report
 - c. Permission for Alcohol/Drug Determination (DUI/D only)
 - d. Record of Sampling for Alcohol Determination (DUI/D only). **THE SPECIMEN WAS SEALED BY LINE MUST BE SIGNED.**
 - e. Submitting Agency
 - f. Analysis Request (NOTE: Alcohol will always be tested in blood samples.)
 2. If a DRE evaluation has been completed, please fill in the DRE Opinion Section.
 3. Fill out the seals at the bottom of the page.
 4. Put the long seals over the filled blood tubes, placing the circle on the top of the tube.
 5. Invert or otherwise mix powder in tube with blood according to the manufacturer's instructions.
 6. Put the short seals over two (2) seams on the Styrofoam packaging.
 7. Place the completed paperwork (**yellow is for agency**) in the mailer along with the samples in the Styrofoam.
 8. Store at less than 8°C/46°F. Mail or deliver to ChemaTox promptly.
-

Drug Panels

5 – Panel:	Benzodiazepines, Cocaine, Methamphetamine, Opiates, Zolpidem
7 – Panel:	Benzodiazepines, Cannabinoids, Cocaine, Methamphetamine, Opiates, Oxycodone, Zolpidem
11 – Panel:	Barbiturates, Benzodiazepines, Cannabinoids, Carisoprodol, Cocaine, Methadone, Methamphetamine, Opiates, Oxycodone, Tramadol, Zolpidem
DFSA:	Alcohol, GC/MS Scan, LC-MS/MS Scan for Benzodiazepines

Other Drug Screens

Drug	Brand Name
Amphetamine	Adderall, Dexedrine, Methamp metabolite
Buprenorphine	Suboxone, Subutex
Fentanyl	Durogesic, Actiq
TCAs (Tricyclic Antidepressants)	Cyclobenzaprine/Flexeril

CERTIFICATION

This kit meets all applicable Colorado State Board of Health regulations, 5 CCR-1005-2, pursuant to 42-4-1304 CRS, 1994 as amended, as of date of manufacture.

BLOOD KIT CONTENTS

- (2) Sterile, Evacuated Tubes with 100mg sodium fluoride and 20mg potassium oxalate. Nominal 10ml draw.
- (1) Sterile Needle
- (1) Towellete containing Aqueous, Non-volatile Antiseptic
- (1) Needle Holder
- (2) Tamper-Evident, Pre-numbered Tube Seals
- (2) Tamper-Evident, Pre-numbered Styrofoam Seals
- (1) 2-part Consent Form
- (1) Mailing container